
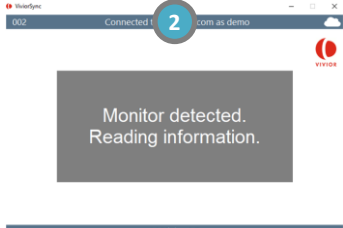
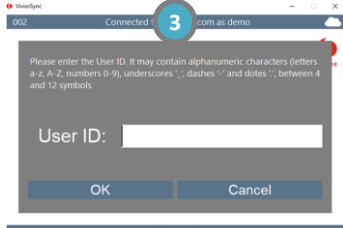
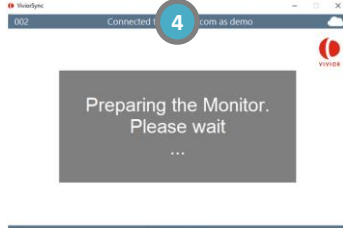
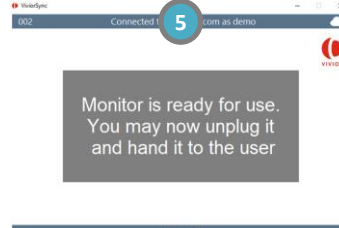





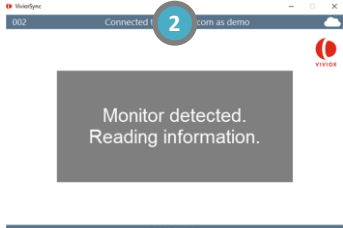

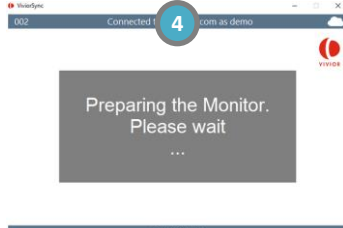
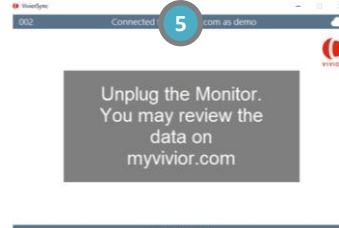
Vivior Monitor

Quick Reference Guide for ViviorSync app

Initialize the Monitor before giving it to a new user

 <p>1</p>	 <p>2</p>	 <p>3</p>	 <p>4</p>	 <p>5</p>
<p>Open the ViviorSync app. Plug the Monitor in your PC (Internet connection is required).</p>	<p>The data on the Monitor is being read by the app, please wait a few seconds.</p>	<p>Enter the User ID (up to 12 characters; using only alphanumeric, “.”; “-” and “_”). Click on “OK”.</p>	<p>The Monitor is being processed by the app, please wait a few seconds.</p>	<p>The Monitor is ready for use. Please unplug it and give it to the user.</p>

Upload the data collected by the Monitor

 <p>1</p>	 <p>2</p>	 <p>3</p>	 <p>4</p>	 <p>5</p>
<p>Open the ViviorSync app. Plug the Monitor in your PC (Internet connection is required)..</p>	<p>The data on the Monitor is being read by the app, please wait a few seconds.</p>	<p>The data collected by the Monitor is automatically being uploaded to the Vivior cloud. Please wait a few minutes.</p>	<p>The Monitor is being processed by the app, please wait a few seconds.</p>	<p>All the data has been erased from the Monitor. You can unplug it. Go to myvivior.com to review the data.</p>